

ConvaQuip Ind., Inc.

Assembly Instructions for Model 1000F Freestanding Trapeze Toll 800-637-8436 Fax 325-677-7217



Tools needed:Rubber Mallet 3/4" Wrench & Drill



Parts: 2 Wheels, 2 Bolt Axles 4 Levelers,4 Self Tapping Screws and Socket Cap Screw



Lay base down as pictured and bolt on both wheels securely.




Screw each leveler all the way in. Do not over tighten.



Set base in upright position.



Align button on each leg and insert into top opening.

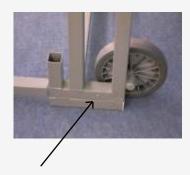

Make sure button secures in place.



Insert M-Shaped part into the 3 slots. Adjust all levelers so they touch floor.



Use rubber mallet to tap down until it bottoms out.



Using drill and socket cap screw provided, screw in the 4 self tapping screws.



Insert Boom into slot on top of the M-Shaped piece.
Make sure it bottoms out.



You can change positions of the boom by lifting pin



Unscrew the Quick-Link and hook over the Boom slider, then screw Quick-link closed



Trapeze is designed to tilt and roll. It is recommended to remove the boom prior to moving.



Trapeze shown in most compact configuration



Easiest configuration for transporting from room to room.